


Sub-pectoral

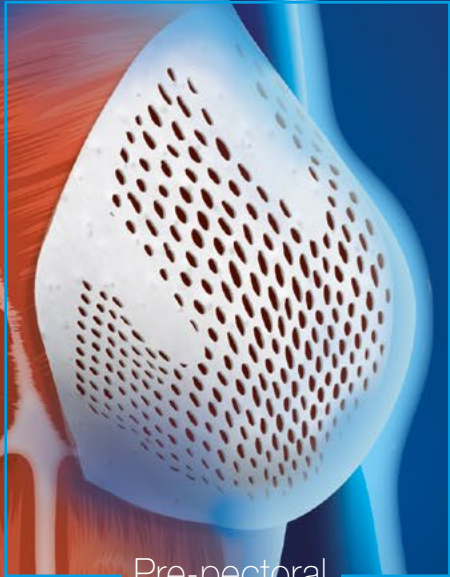

Pre-pectoral

ACELLULAR DERMAL MATRIX
**FOR BREAST
RECONSTRUCTION**

SURGERY



cellis[®]
Breast

TISSUE REGENERATION SUPPORTING BREAST RECONSTRUCTION

Meccellis Biotech is an international company recognized as an expert in tissue and cell engineering.

Meccellis Biotech aims to certify high quality medical devices so that health professionals can concentrate on providing the best patient care.





Meccellis Biotech has developed a new range of acellular dermal matrix to reinforce soft tissue for breast reconstruction. CELLIS® Breast is a surgical membrane made of porcine dermis.

Pocket

CELLIS® Breast POCKET FOR PRE-PECTORAL BREAST RECONSTRUCTION


CELLIS® BENEFITS

- + Quick hydration,
- + Stretchable,
- + Easy to handle and place,
- + Designed with fenestrations to help fluid drainage,
- + Adapted shapes, allow total cover for much larger implants,
- + Surgery performed quicker, the pectoralis muscle not being dissected,
- + Freeze-dried,
- + Free from preservative.

Sizes	References	Thicknesses	Shapes
18 x 25 cm	CB1825FS	0,9 mm	 SHORT
18 x 25 cm	CB1825FL	0,9 mm	 SHORT
18 x 40 cm	CB1840FS	0,9 mm	 LONG
18 x 40 cm	CB1840FL	0,9 mm	 LONG





GUIDELINE FOR

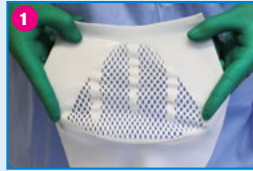

Hydrate for a minimum of 15 minutes in a saline solution





SHORT

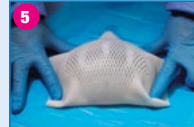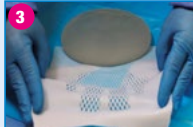


LONG



Cut the matrix to the shape needed

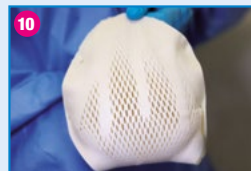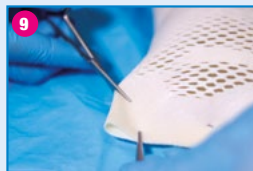


The matrix is ready to be fixed onto the pectoralis muscle

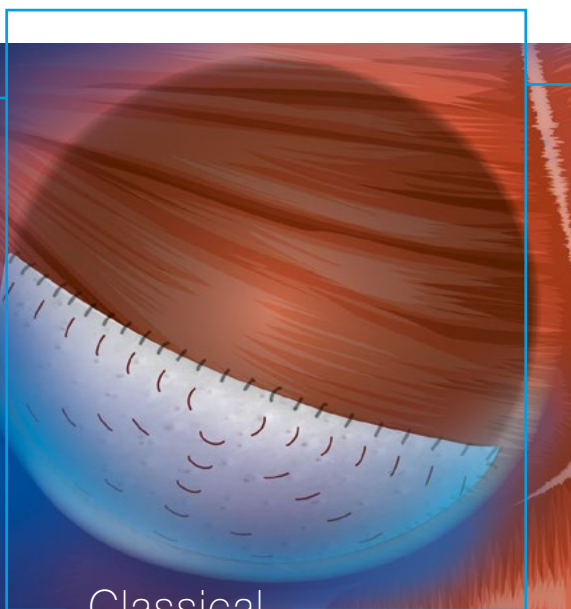

Classical

CELLIS® Breast FOR SUB-PECTORAL BREAST RECONSTRUCTION

Sizes	References	Thicknesses	Shape
12 X 18 cm	CB1218F	0,9 mm	
12 X 24 cm	CB1224F	0,9 mm	



- + Quick hydration,
- + Easy to handle and place,
- + Designed with fenestrations to help fluid drainage,
- + Adapted shapes and sizes available,
- + Freeze-dried,
- + Free from preservative.



PRODUCT DESCRIPTION

CELLIS® Breast is a cell-free, non-pyrogenic collagen matrix (acellular dermal matrix ADM) obtained from porcine skin. CELLIS® Breast is intended for use as a surgical membrane in soft tissue repairs and serves to support, cover or replace tissue. CELLIS® Breast is available in various sizes, shapes and thicknesses. CELLIS® Breast comes in double sterile packaging, is supplied dry and does not contain any preservatives. This surgical membrane is a resilient, biocompatible implant, which incorporates into the host tissue through cellular and microvascular infiltration.

COMPOSITION

Sterile, acellular, type I & III porcine dermis derived collagen matrix.

INDICATION

CELLIS® Breast is intended for implantation to reinforce soft tissue where weakness exists and for surgical repair of damaged or ruptured soft tissue membranes.

It's indicated to be used to reconstruct, to recontour and to reform the host's human soft connective tissue particularly where loss of tissue has occurred and as a supporting tissue in surgical procedures such as breast reconstruction.


CELLIS® Breast is intended for implantation for breast reconstruction after mastectomy.

CONTRAINDICATIONS

The surgical membrane is made of porcine dermis and cannot be used in patients with known hypersensitivity to porcine materials.

STORAGE

- Store at room temperature as indicated on the label.
- Keep away from heat sources and direct sunlight.
- Store inside the original packaging.

Class III medical device:  1434
August 2018

Brochure is intended for healthcare professionals only.
The « Instructions for Use » attached to the packaging should be read carefully.

Visit
www.meccellis.com
and contact your sales representative for more information.



MECCELLIS BIOTECH

75, rue de Québec
17000 La Rochelle
FRANCE

Tél. : +33 (0)5 17 81 07 63

contact@meccellis.com
www.meccellis.com



Distributed by